



## Installation Manual

### Introduction

Welcome to the FONIX NOAH Audiometer Module. This program lets you collect test results from your audiometer and control it remotely from your computer. The FONIX FA10 and FA12 are supported.

### Requirements

- NOAH version 3.7 or 4.0 and computer requirements for running NOAH
- Windows XP Professional SP2 (32-bit), Windows Vista (32-bit versions) including: Vista Home Basic, Vista Home Premium, Vista Business, Vista Enterprise, Vista Ultimate, Windows 7 (32 and 64-bit)
- 1024 x 768 video resolution
- 1 free serial port or USB port
- Your FONIX audiometer must have RS232 functionality

### Setup

The NOAH Audiometer Module installation is a two part process. In the first part, you install the module on your computer using the setup program that comes on the installation disk. **Before you start the installation process, make sure that NOAH is already installed and functioning on your computer.** Next, you must enter the serial number and registration code, obtained upon purchase, to enable saving of your data.

The setup process is fairly straightforward, but it does require some basic understanding of Windows. Your local computer support person should be able to help you with most questions, but you may also contact us at [support@frye.com](mailto:support@frye.com) if you need any further help.

## Part 1: Running the setup program

The first part of the installation process is running the setup program that comes with your NOAH Audiometer Module installation disk. You will need administrator rights on your computer if you are running Windows Vista or Windows 7.

1. Close all programs on your computer.
2. **Remove any existing version of the FONIX NOAH Module or NOAH Audiometer Module by using the Add/Remove Programs function on the Windows Control Panel.**
3. Insert the NOAH Audiometer CD into your PC's CDRom drive. This should automatically start the setup program. Otherwise, use Windows Explorer to open the contents of the installation disk and double-click on setup.exe.
4. Follow the instructions that appear in the installation wizard. This mostly involves clicking the **Next** button and filling in some basic information.
5. Wait for the installation wizard to complete. This may take several minutes. There will be a prompt asking you whether you want to install the program into NOAH. Click **Yes**.

In some cases, you may encounter some errors in the installation process. Try ignoring them and continuing with the installation. Usually the installation will still be successful.

## Part 2: Entering your registration code

When the NOAH Audiometer Module is installed, the Save function is disabled until the serial number and registration code have been entered. This allows us to give our free demo copies of our software program; users who install the program without purchasing the registration code and serial numbers cannot save records.

If you have already purchased the NOAH Audiometer Module, the serial number and registration code should be printed on a sticker on the outside of the CD installation case. If you have obtained a free copy of the NOAH Audiometer Module for demonstration purposes, you can obtain the serial number and registration code by purchasing the NOAH Audiometer Module from your local FONIX distributor or from the Frye factory.

**To register your product:**


1. Open the NOAH Audiometer Module.
2. Open the **Help** menu and select **Register**.
3. Enter the **Serial Number** and the **Registration Code** provided on the NOAH Audiometer Module Installation CD. If no such number is present, contact Frye to purchase the product.
4. Click **Register** to complete the registration and click **Exit** to close the registration window.

You do not have to transmit any information back to Frye Electronics to register your product.


If you are installing the NOAH Audiometer Module on a Windows Vista or Windows 7 computer, you may need to run NOAH as administrator during the registration process. To do this, right-click on the NOAH icon and select **Run as administrator**.

## Opening the NOAH Audiometer Module

The following are instructions on how to open the NOAH Audiometer Module:

1. Open NOAH.
2. Open a client file.
3. Click the **Module Selection** button. 
4. Select the **Measurement** tab.
5. Select the **NOAH Audiometer Module** icon. This will open the NOAH Audiometer Module window.

## Assigning the NOAH "Open Audiogram Module" button

You can open the NOAH Audiometer Module with one click by assigning the NOAH "Open Audiogram Module" button  to the NOAH Audiometer Module. To do this:

1. Open NOAH.
2. In the Client Register, open the **Tools** menu and select **Module Buttons**.
3. Use the drop-down menu to select **FONIX Audiometer Module** for the **Audiogram** button.
4. Click **OK**.

For more information on operating the NOAH Audiometer Module, see the electronic help files by opening the **Help** menu and selecting **Help** or by pressing your F1 key.

## Contact

If you have any questions on the installation or operation of the NOAH Audiometer Module, you can contact us at:

Frye Electronics Inc.  
9826 SW Tigard St.  
Tigard, OR 97223  
USA

Phone: (503) 620-2722 or (800) 547-8209

Fax: (503) 639-0128

Email: [support@frye.com](mailto:support@frye.com)



**FRYE ELECTRONICS, INC.**

P.O. Box 23391 • Tigard, OR 97281-3391 • USA

(503) 620-2722 • (800) 547-8209

Fax: (503) 639-0128

[www.frye.com](http://www.frye.com) • email: [sales@frye.com](mailto:sales@frye.com)