



## Installation Manual

### Introduction

Welcome to the FONIX NOAH Real-ear Module. This program lets you remotely control your FONIX hearing aid analyzer from your computer and compare your results to the DSL mI/O, NAL-NL1, or MOD NAL targets. The FONIX 8000, 7000, FP35, and FP40 hearing aid analyzers are supported.

### Requirements

- NOAH version 3.7 or 4.0 or newer and computer requirements for running NOAH
- Windows XP Professional SP3 (32-bit), Windows Vista (32-bit versions) including: Vista Home Basic, Vista Home Premium, Vista Business, Vista Enterprise, Vista Ultimate, Windows 7 (32 and 64-bit)
- 1024 x 768 video resolution
- 1 free serial port or USB port
- FONIX Analyzer: 8000 (v2.10), 7000 (v1.70), FP40 (v3.70 and RS232 Option), or FP35 (v6.30). Some functions may work on older software versions and on the 6500-CX (v4.91 and RS232 Option), but they have not been tested. All analyzers require the Real-ear Option to perform real-ear measurements.

### Setup

**Before you start the installation process, make sure that NOAH 3 or NOAH 4 is already installed and functioning on your computer.** The NOAH Real-ear Module installation is a two part process. In the first part, you install the module on your computer using the setup program that comes on the installation disk. Next, you must register the program to allow unlimited use with the connected hearing aid analyzer.

The setup process is fairly straightforward, but it does require some basic understanding of Windows. Your local computer support person should be able to help you with most questions, but you may also contact us at [support@frye.com](mailto:support@frye.com) if you need any further help.

## Part 1: Running the setup program

The first part of the installation process is running the setup program that comes with your NOAH Real-ear Module installation disk:

1. Close all programs on your computer.
2. **Remove any existing version of the FONIX NOAH Module or NOAH Real-ear Module by using the Add/Remove Programs function in the Windows Control Panel.**
3. Insert the NOAH Real-ear CD into your PC's CDROM drive. This should automatically start the setup program. Otherwise, use Windows Explorer to open the contents of the installation disk and double-click on setup.exe.
4. Follow the instructions that appear in the installation wizard. This mostly involves clicking the **Next** button and filling in some basic information.
5. Wait for the installation wizard to complete. This may take several minutes.
6. At the end of the installation process, a window will appear, asking if you want to install the program into NOAH. Click **Yes**.

In some cases, you may encounter some errors in the installation process. Try ignoring them and continuing with the installation. Usually the installation will still be successful.

## Part 2: Entering your registration key

The FONIX NOAH Real-ear Module uses registration keys tied to specific hearing aid analyzers. You will only be able to perform a limited number of measurements with a non-registered hearing aid analyzer.

To obtain a registration key for a specific hearing aid analyzer, you will first need to determine the analyzer's "instrument access code." You must then contact the Frye factory and exchange this access code for the registration key for that analyzer.

**To register your FONIX hearing aid analyzer:**


1. Open the NOAH Real-ear Module.
2. Open the **Help** menu and select **Product Registration**.

3. Make sure your hearing aid analyzer is connected to your computer. See the help files for more details.
4. Click **Get Instrument Access Code**. This will open a window with the access code for the connected analyzer.
5. Contact your FONIX distributor or the Frye factory with the instrument access code. They will then be able to provide you with the corresponding registration key for that analyzer.
6. Enter the registration key into the Product Registration window. You can enter up to five different keys. This is for customers who use more than one analyzer with the same computer.
7. Click **Update Registration Keys**. This will confirm the registration key is valid for the connected analyzer. Your program is now registered.


When installing the FONIX NOAH Real-Ear Module on Windows Vista or Windows 7 computer, you may need to run NOAH as administrator to complete the registration process. To do this, right-click on the NOAH icon and select **Run as administrator**.

## Opening the NOAH Real-ear Module

The following are instructions on how to open the NOAH Real-ear Module:

1. Open NOAH.
2. Open a client file.
3. Click the **Module Selection** button. 
4. Select the **Measurement** tab.
5. Select the **NOAH Real-ear Module** icon. This will open the NOAH Real-ear Module program.

## Assigning the NOAH “Open Real-ear Measurement Module” button

You can open the NOAH Real-ear Module with one click by assigning the NOAH “Open Real-ear Measurement Module” button  to the NOAH Real-ear Module. To do this:

1. Open NOAH 3.
2. In the Client Register, open the **Tools** menu and select **Module Buttons**.
3. Use the drop-down menu to select **FONIX NOAH Real-ear Module** for the **Real-ear Measurement** button.
4. Click **OK**.

For more information on operating the NOAH Real-ear Module, see the electronic help files by opening the **Help** menu and selecting **Help** or by pressing your F1 key.

## **Contact**

If you have any questions on the installation or operation of the NOAH Real-ear Module, you can contact us at:

Frye Electronics Inc.  
9826 SW Tigard St.  
Tigard, OR 97223  
USA

Phone: (503) 620-2722 or (800) 547-8209

Fax: (503) 639-0128

Email: [support@frye.com](mailto:support@frye.com)



### **FRYE ELECTRONICS, INC.**

P.O. Box 23391 • Tigard, OR 97281-3391 • USA

(503) 620-2722 • (800) 547-8209

Fax: (503) 639-0128

[www.frye.com](http://www.frye.com) • email: [sales@frye.com](mailto:sales@frye.com)