

# Assembly Instructions

## Velcro attachment for Probe Microphone Use with Infant Headband

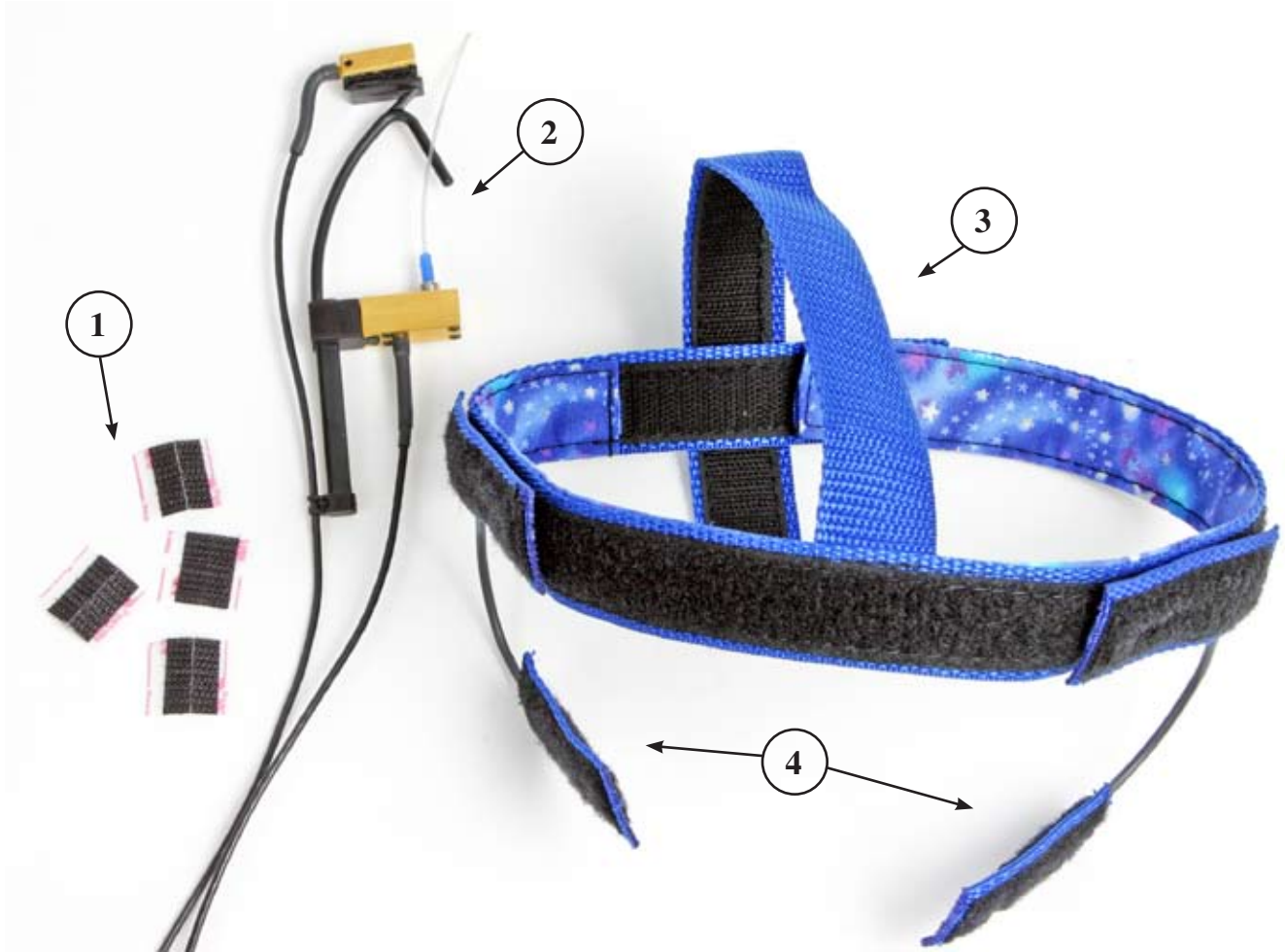


Figure 1

Parts required: See figure 1.

**Item 1** - 3/4" Velcro (P/N 042-1530-00)

**Item 2** - Probe Microphone Assembly (P/N M567 or M767)

**Item 3** - Headband (P/N 042-0534-00 or 042-0535-00)

**Item 4** - Earpieces (P/N 042-0536-00 or 042-0537-00)

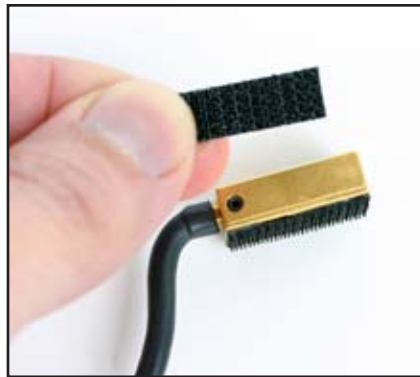


Figure 3



Figure 4

These instructions describe how to modify the integrated probe microphone for use with the infant headband set.

**Step 1:**

Detach Reference Microphone and Probe Microphone from earhook. *See figure 2.*

**Step 2:**

Attach Velcro strip to both sides of Reference Microphone. *See figures 3 and 4.*

*Note: There are two pieces of velcro per 3/4" section provided.*

**Step 3:**

Attach Velcro strip to both sides of Probe Microphone. *See figure 5.*

**Step 4:**

Attach Microphones to Headband Earpieces. *See figure 6.* Velcro is attached to both sides of microphones to allow testing of either left and/or right ear using one probe microphone assembly.

**Notes:**

For calibration of Probe Microphone assembly, it will be required to remove the Velcro from sides of reference microphone. Additional Velcro material is provided to allow for multiple applications.

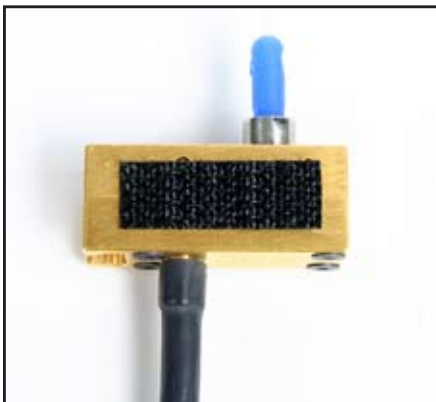


Figure 5



Figure 6



**FRYE ELECTRONICS, INC.**  
 P.O. Box 23391 • Tigard, OR 97281-3391 • USA  
 (503) 620-2722 • (800) 547-8209  
 Fax: (503) 639-0128  
 www.frye.com • e-mail: sales@frye.com