



3.0v Load Fonix 8000 Printer Paper

2.4.4 Loading the thermal paper

1. Open the printer door.
2. Press Lever back till Roller Bar pops loose and drops into lower position against open door.
3. Remove the plastic paper roller from the printer mechanism. If you are replacing an empty paper roll, remove and discard the paper spool from the printer mechanism, but **do not discard the plastic spindle**. The spring bar (white plastic back tension rod), located directly behind the paper roller is not a removable part.
4. Insert the plastic roller into a new paper roll.
5. Install the plastic roller and paper roll into the printer mechanism, with paper facing out and feeding from the top of the roll. The paper roller will fit into the slots on the left and right side of the printer mechanism. The paper roller should drop into position without additional adjustment.

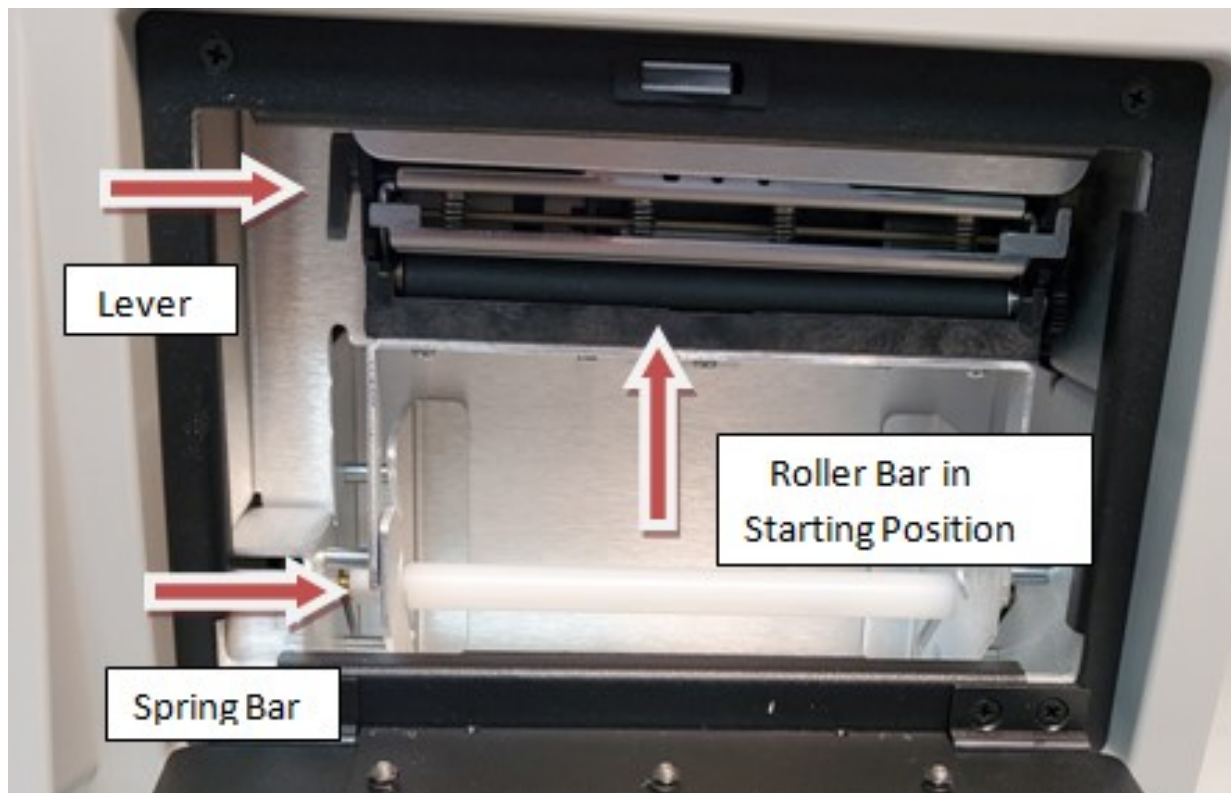

6. Load paper roll so it come off the back of the spindle and drop into the slots on either side of the white bottom spring bar

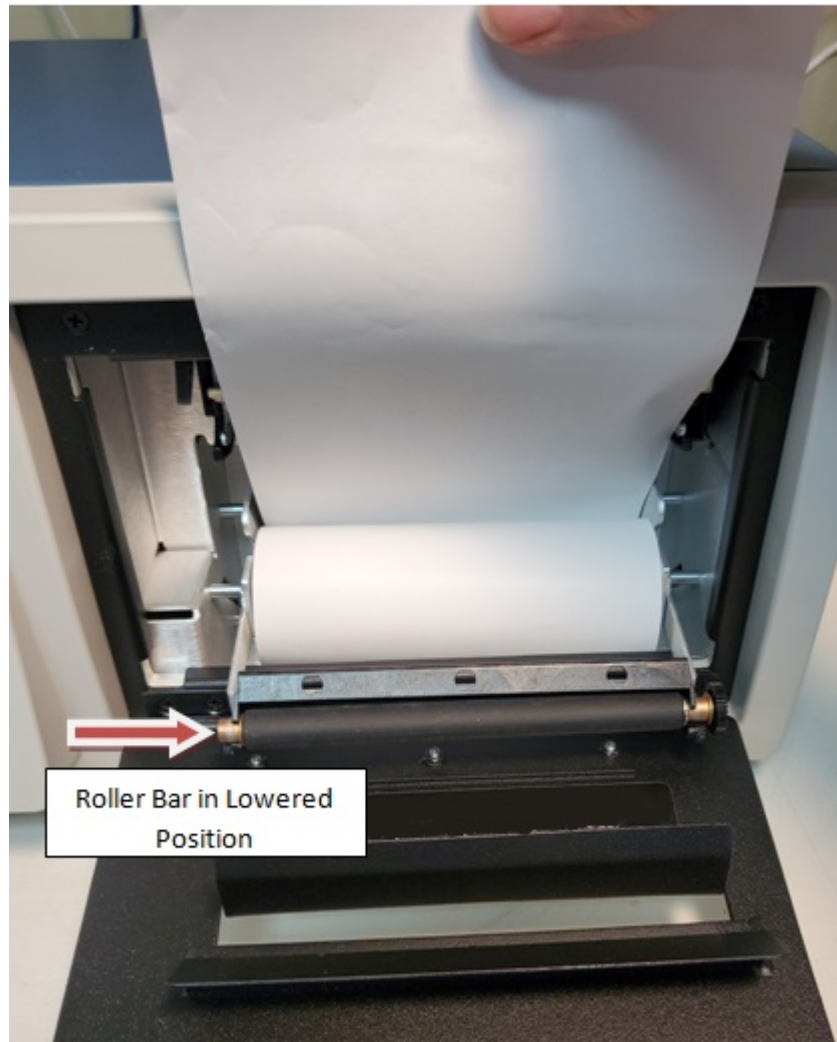

7. Move roller Bar back up and push until it clicks back in to starting position with paper behind it.



8. Feed Paper through door slot, close printer door and tear off extra paper.

