


USB to Serial Cable Installation

This cable is used to connect your FONIX hearing aid analyzer or audiometer to a personal computer.

Installation

In order to install this cable on your Windows-computer, you must have **Administrative** rights to the computer. If you do not have these rights on your user account, please contact your local computer system administrator for assistance with the installation.

1. Insert the USB cable into the personal computer. The computer should automatically detect the USB device and run the New Hardware Wizard.
2. A message will appear, asking, "Can Windows connect to Windows Update to search for software?" Select **No, not this time** and click **Next**.
3. Select **Install from a list of specific location. (Advanced)** and click **Next**.
4. Insert the USB driver installation CD included with the USB cable into your computer's CDROM drive.
5. Select **Search removable media (floppy, CD-ROM...)** and click **Next**.
6. When the following message appears, click **Continue Anyway**.

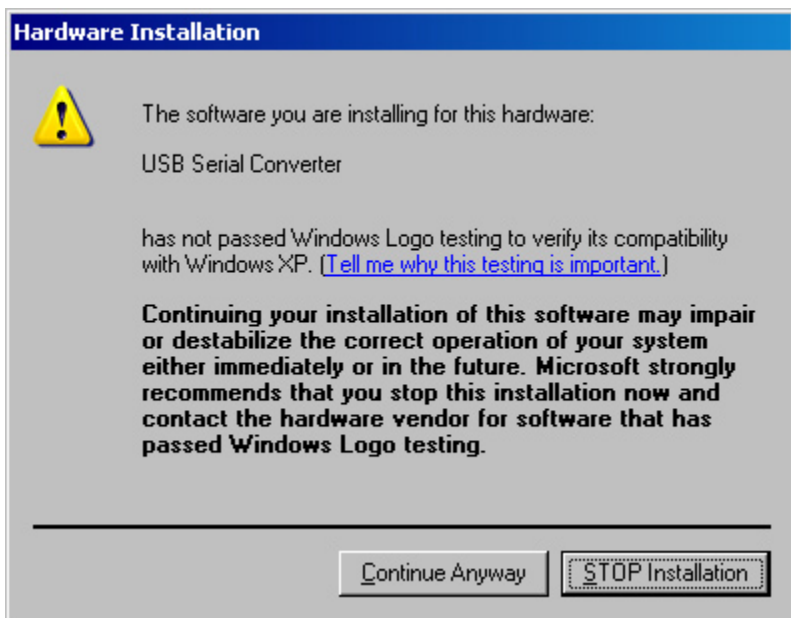

USB to Serial Cable Installation (continue)

7. Click **Finish**.
8. The New Hardware Wizard will run again. Repeat steps 2-5 above. When the following message appears, click **Continue Anyway**.
9. The USB cable installation should now be complete.

